

ClickZip Needle Retractable Safety Syringe

ClickZip™ is the future of needle safety

ClickZip™ prevents reuse and reduces needlestick injury due to its innovative tilted needle technology that encapsulates the needle in the syringe barrel.



Scan this code for
3D instructions





Description of technology

The innovative ClickZip™ range includes the ClickZip™ Needle Retractable Safety Syringe featuring a high-quality, triple bevel needle, designed for increased patient comfort. The unique design also ensures minimal leakage and medicine wastage.

With community health being a priority focus for Numedico, the retractable needle injectable delivery system also improves safety by reducing needlestick injuries and prevents needle reuse.

ClickZip™ meets international standards for medical devices and OSHA recommendations for needlestick safety devices.*

* Full compliance with ISO Quality System for Manufacturing

* Meets NIOSH recommendation for needlestick devices




Features & Benefits

- Highest quality Japanese designed stainless steel cannulas
- Latex free
- No leakage
- No gap – minimal dead space
- Active-retractable needle technology – manual retraction
- After the injection is given, the user can retract the needle before or after removing the needle from the patient
- No change to injection technique
- When retracted after use, the needle tilts to one side into the syringe barrel
- The barrel incorporates the needle back-stopper
- The user can hear and feel when the locking mechanism is activated



Instructions for use

The ClickZip™ Needle Retractable Safety Syringe is to be used for intra-muscular or subcutaneous injection of medications into a patient and is intended to prevent needle stick injuries.

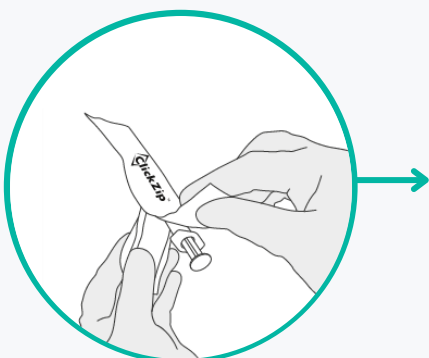

STEP 1: Open

- > Follow hygiene & PPE processes.
- > Open paper wrap from the top by separating the paper and peeling downwards. (Please discard if packaging is damaged.)
- > Remove safety cap.




STEP 2: Draw

- > Ensure the needle is fixed tight.
- > Draw out the medication.
- > Use a standard aseptic technique to fill the syringe.




STEP 3: Inject / 'Click'

- > Inject the medication by fully depressing the plunger to engage the locking mechanism.
- > User should feel and hear a click.

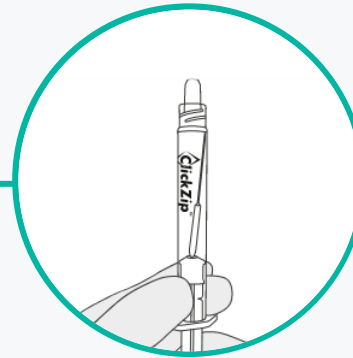

STEP 4: 'Zip'

- > Remove (zip) the plunger back to safely retract the needle into the barrel.
- > The needle will tilt to one side to prevent needlestick injury and reuse.

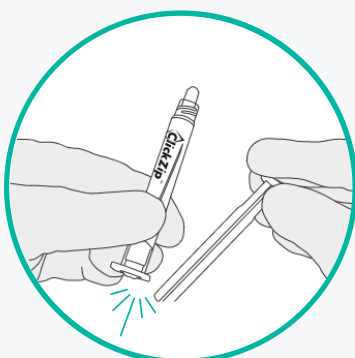

STEP 5: 'Snap'

- > Snap off the plunger at the breaking point.
- > Discard the complete unit as per your policy.


WARNING

To avoid early activation of the safety mechanism **DO NOT** depress the plunger to the end.



Product Structure of the ClickZip™ technology

ClickZip™ Needle Retractable Safety Syringe is a single use sterile needle retractable safety syringe (RUP Syringe) that is designed to draw the needle up into the syringe barrel to reduce needlestick injury and prevent reuse.

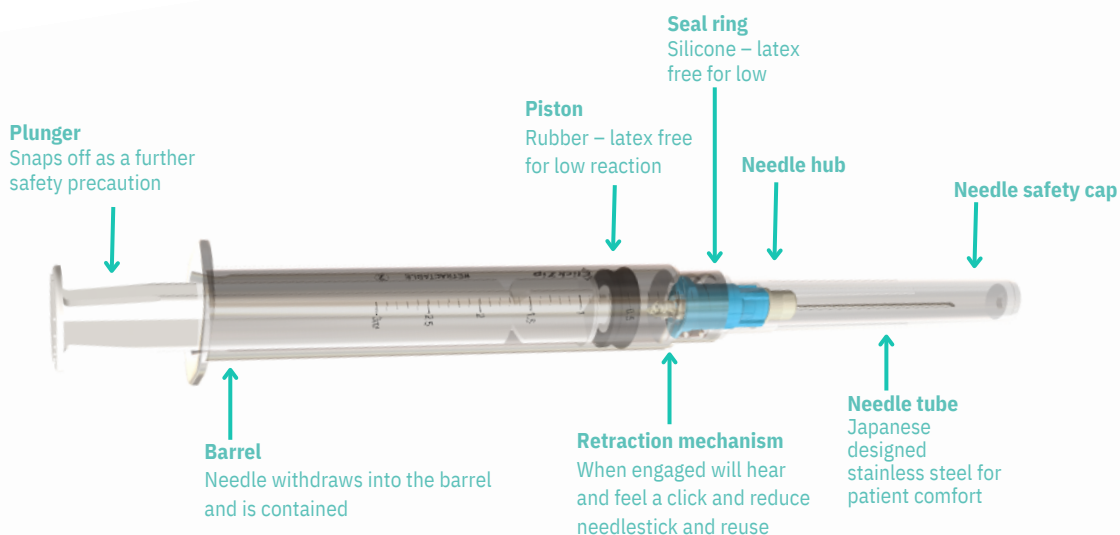

For more details and for full specifications contact:
sales@numedico.com.au

Numedico Technologies Pty Ltd
Registered office
Level 21, 91 King William Street SA 5000
AUSTRALIA sales@numedico.com.au

Scan this code for
3D instructions



Providing quality medical supplies globally from a trusted Australian brand.

www.numedico.com